




CANADIAN
MADE

NOTATIONS:

SPECIAL ORDER

1. Quantities listed are not guaranteed until time of arrival
2. Pricing is subject to change without notice
3. No warranties or claims are applicable to these products
4. Sold as is, no returns or credits
5. Canadian made

Materials Manufacturer registration # MDEL 14031.
Bob Dale Gloves and Imports Ltd. makes no claims on manufacturer registrations.

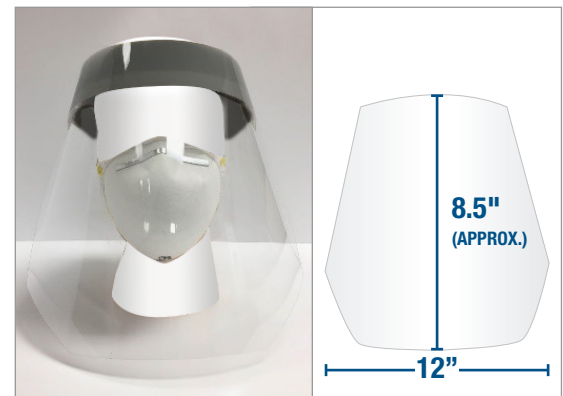

Size	Part #	Distributor Part #	Qty
One Size	88-1-800		

Pack size: 30 Case size: 240 Ven pack: Yes No

FEATURES & BENEFITS

	Clear Poly Shield .007 Polyethylene Terephthalate		Low Distortion & Anti-Fogging Coated
	Soft Foam Headband		Splash Protection
	Fully Adjustable Head Straps		Lightweight, Comfortable & Reusable

BASIC CLEANING AND DISINFECTING



**Hand Wash Warm Water
+ Mild Cleaning Agent**

To avoid creating surface scratches, submerge the face shield in warm water to dislodge particulate matter. Adding a mild liquid dish, like Dawn, reduces surface tension, and allows the soapy water to evenly disperse across the surface. A soft cloth or sponge can be used to gently clean the shield. Rinse the shield in clear water and dry with a soft cotton towel or a microfiber cloth. Remember soap and water removes microbes on our hands, so it will work on your shield as well.

Some clinicians also wish to use a surface disinfectant wipe or spray, but these products can leave a visible residue, which then needs to be removed. Time also matters when it comes to disinfectant solutions. Strictly follow the specific disinfectant directions. While a quick wipe may seem efficient, the contact time must be followed. The required wet contact time can range from thirty seconds to as long as four minutes. A 70% alcohol wipe (Isopropanol) will also disinfect and keep plastic surfaces clean.

Revision date: November 11, 2020